AMT Introducer

For use with the MINI™ Button

CE
2862

- ES** **AMT Estilete** - Para usar con el botón globo MINI™
- DE** **AMT Stilet-Katheter** - Zur Verwendung mit MINI™ Ballon Sonde
- IT** **AMT Mandrino** - Da utilizzare con il bottone MINI™
- FR** **AMT Stylet** - Pour utilisation avec bouton MINI™
- NO** **AMT Sonde** - Til bruk sammen med MINI™ knapp
- SV** **AMT Mandrang** - För användning med MINI™-knapp
- NL** **AMT-Introducer** - Voor gebruik met de MINI™ knop
- DA** **AMT Introducer** - Til brug sammen med MINI™ Button
- FI** **AMT Sisäänviejä** - Käytettäväksi MINI™ Button -napin kanssa
- AR** **AMT** - مداخل للاستخدام مع MINI™ Button
- CS** **Zaváděč AMT** - K použití s knoflíkem MINI™
- EL** **AMT Εισαγωγέας** - Για χρήση με το MINI™ κουμπί
- ET** **AMT Sisesti** - Kasutamiseks MINI™ Buttoniga
- HU** **AMT Bevezető** - A MINI™ Button használatához
- JA** **AMTインテロデューサ** - MINI™ ボタンとの併用
- PL** **Introduktor AMT** - Przeznaczony do użytku z urządzeniem MINI™ Button
- PT-BR** **Introduitor AMT** - Para uso com o Botão MINI™
- RO** **Dispozitiv de Introducere AMT** - A se utiliza cu butonul MINI™
- SL** **Uvajalo AMT** - Za uporabo s pripomočkom MINI™ Button
- TR** **AMT Introduşer** - MINI™ Buton İle Kullanılır
- LV** **AMT Ievadītājs** - Lietošanai ar MINI™ Button ierīci
- BG** **Устройство за Въвеждане AMT** - За използване с бутон MINI™
- LT** **AMT Introduiseris** - Naudoti su MINI™ sagute
- PT** **Introduitor da AMT** - Para utilização com o Botão MINI™
- SK** **Zavádzač AMT** - Na použitie s tlačidlom MINI™

R_x ONLY

MD

C3805-H 04/2024

Made in U.S.A.

STERILE | **EO**

AMT Introducer

For use with the MINI™ Button

DIRECTIONS FOR USE

Caution: Federal (USA) law restricts this device to sale, distribution, and use by or on the order of a physician.

CONTENTS:

- (1) Introducer Clamp
- (1) Stylet

INTENDED USE:

The AMT Introducer is an effective tool intended to be used to stiffen the MINI™ Button for easier device placement. The AMT Introducer is intended to be used by clinicians for patients of all ages (infants to elderly). The device is especially useful in patients with stoma tracts that are long, tight or angled.

INDICATIONS FOR USE:

The introducer is indicated to assist in the placement of AMT MINI™ Button feeding devices by providing temporary feeding tube stability during insertion through well established patient stoma tract.

CONTRAINDICATIONS FOR USE:

1. Lack of adherence of the stomach to the abdominal wall
2. Lack of a well established gastrostomy site
3. Evidence of infection
4. Uncertainty as to gastrostomy tract direction and length(abdominal wall thickness)

COMPLICATIONS:

No complications exist at this time for the AMT Introducer.

NOTE: Please contact AMT, our Authorized Representative (EC Rep), and/or the competent authority of the member state in which you are established if a serious incident has occurred in relation to the device.

CLINICAL BENEFITS:


Clinical Benefits to be expected when using the AMT Introducer include but are not limited to:

- Provides stiffening support to the MINI™ Button for easier placement
- Ideal for patients with long, tight, or angled stoma tracts
- Increased ease of placement reduces pain from replacing the device, ensuring the patient can receive proper nutrition and medication

PERFORMANCE CHARACTERISTICS:

Performance Characteristics of the AMT Introducer include but are not limited to:

- Provides ease of placement of MINI™ Button
- Stylet and Clamp can be repositioned, if necessary
- Compatible with devices 12F and larger
- Compatible with devices up to 6.5 cm in length

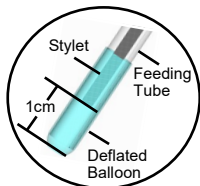Not to scale

DIRECTIONS FOR USE:

NOTE: For use with the AMT MINI™ Button feeding device only. Refer to the directions for use supplied with the MINI™ Button for proper placement/use of the feeding device.

CAUTION: Prior to use, please inspect all contents for damage. If the package is damaged or sterile barrier is breached, do not use the product.

1. Insert the Introducer Clamp into the MINI™ Button feeding port.
2. Insert the Stylet through the Introducer Clamp and MINI™ Button until the tip is 1cm from the distal tip of the MINI™ Button.
3. Lock the Stylet in place by squeezing the Introducer Clamp shut. Verify the Stylet placement length after clamping. Unclamp and reposition if necessary.
4. Prepare the patient for device insertion as specified in the MINI™ Button's directions for use.
5. Apply lubricant to the stoma site and balloon tip.
6. Insert the MINI™ Button as specified in the device's directions for use. **Discontinue Introducer use if resistance is encountered during MINI™ Button insertion.**
7. Once the MINI™ Button is fully inserted, unclamp the Introducer Clamp and remove the Stylet. Discard upon removal. **To avoid damaging the MINI™ Button's internal valve, always remove the Stylet before the Introducer Clamp.**
8. Gently remove the Introducer Clamp while holding the MINI™ Button's external bolster against the patient to prevent inadvertent feeding device removal. Discard clamp upon removal.
9. Inflate the retention balloon to the prescribed volume and gently pull on the feeding device to observe retention.

WARNING: THIS DEVICE IS INTENDED FOR SINGLE USE. DO NOT REUSE, REPROCESS OR RE-STERILIZE THIS MEDICAL DEVICE. DOING SO MAY COMPROMISE BIOCOMPATIBILITY CHARACTERISTICS, DEVICE PERFORMANCE AND/OR MATERIAL INTEGRITY; ANY OF WHICH MAY RESULT IN POTENTIAL PATIENT INJURY, ILLNESS AND/OR DEATH.

NOTE: The device can be disposed of by following local disposal guidelines, facility protocol or through conventional waste.

THANK YOU! Thank you for choosing AMT. For additional help and information regarding the use of our device, feel free to contact AMT with the contact information on the back of the instructions for use. We are happy to hear your thoughts and help with your concerns and questions.

 Prescription Only	 Sterilized Using Ethylene Oxide
 Not made with natural rubber latex	 Not made with DEHP (Di(2-ethylhexyl) Phthalate)
 Do not resterilize	 Do not use if package is damaged and consult instructions for use
 Single Use Only	 Medical Device

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Applied Medical Technology, Inc.