AMT Guidewire Compatible Balloon Stoma Measuring Device

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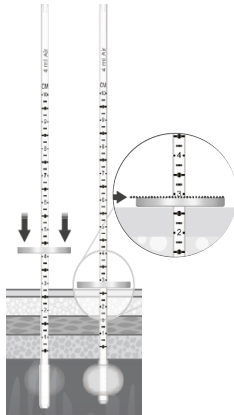
Made in the U.S.A.

C4366-F 04/2024

CAUTION: Federal (USA) law restricts this device to sale, distribution, and use by or on the order of a physician.

INTENDED USE: The Guidewire Compatible Balloon Stoma Measuring Device is intended to be used as an effective tool to measure the length of a gastrointestinal stoma. It is also compatible with a guidewire. The Guidewire Compatible Balloon Stoma Measuring Device is intended to be used for patients of all ages (infant to elderly).

INDICATIONS FOR USE: The AMT Balloon Stoma Measuring Devices are to be used by medical professionals and home health givers in association with replacement gastrostomy feeding devices. The purpose of the AMT Balloon Stoma Measuring Devices is to aid in determining the depth (length) of an established stoma tract in order to assist in determining the proper length replacement gastrostomy feeding device to be used. Minimum French Size is 10FR.

CONTRAINDICATIONS FOR USE: At this time there are no contraindications associated with the use of the Guidewire Compatible Balloon Stoma Measuring Device.

COMPLICATIONS: At this time there are no complications associated with the use of the Guidewire Compatible Balloon Stoma Measuring Device.

NOTE: Please contact AMT, our Authorized Representative (EC Rep), and/or the competent authority of the member state in which you are established if a serious incident has occurred in relation to the device.

CLINICAL BENEFITS: Clinical Benefits to be expected when using the Guidewire Compatible Balloon Stoma Measuring Device include but are not limited to: Increases the probability that a patient will receive a properly sized low profile gastrostomy tube allowing for optimal channeling of nutrition and medication • Reduces potential formation of granulation tissue and other skin issues resulting from feeding tube which is too tight or too loose/leaking • Reduces potential of inadvertent tube dislodgement which would necessitate replacement • Guidewire compatibility helps to ensure the ease of proper placement of low profile feeding devices

PERFORMANCE CHARACTERISTICS: Performance Characteristics of the Guidewire Compatible Balloon Stoma Measuring Device include but are not limited to: Provides a means of accurately measuring the stoma tract length • Helps to ensure proper fit, comfort, and safety of replacement gastrostomy feeding device • Compatible with stoma tracts of varying diameter and length corresponding to common sizes of gastro-intestinally placed feeding tube devices • Guidewire Compatible

WARNING: THIS DEVICE IS INTENDED FOR SINGLE USE. DO NOT REUSE, RE-STERILIZE OR REPROCESS THIS MEDICAL DEVICE. DOING SO MAY COMPROMISE BIOCOMPATIBILITY CHARACTERISTICS, DEVICE PERFORMANCE AND/OR MATERIAL INTEGRITY; ANY OF WHICH MAY RESULT IN POTENTIAL PATIENT INJURY, ILLNESS AND/OR DEATH.

CAUTION: Please inspect all contents of the kit for damage. If the package is damaged or sterile barrier is breached, do not use the product. If any kit component is missing or damaged, discard and obtain another device.

INSTRUCTIONS:

1. Open the Guidewire Compatible Balloon Stoma Measuring Device package and inspect the parts. Verify balloon inflates prior to using. If any damage is noticed, discard and obtain another device for use.
2. Slide the measuring disc up the tube - away from the balloon. If the disc does not slide on the tube, discard and obtain another device for use.
3. Remove the existing feeding tube from patient according to the original device's instructions for use. Work as quickly as possible to avoid spontaneous closure of the stoma.
4. Lubricate the balloon tip of the Guidewire Compatible Balloon Stoma Measuring Device using a water soluble lubricant.

- Gently slide the tube portion of the Guidewire Compatible Balloon Stoma Measuring Device through the stoma and into the stomach.
- NOTE:** A guidewire can optionally be used to aid in guidance through the stoma tract. Measuring Device will slide over guidewires up to 0.038" in diameter.
- Insert a slip tip syringe containing 4ml AIR into the handle side orifice and inflate balloon. Remove the syringe.
 - Pull gently on the Guidewire Compatible Balloon Stoma Measuring Device until you can feel the balloon against the inside stomach wall.
 - Slide the measuring disc down on the tube until it rests on the outside of the abdomen.
 - Read the measurement value on the tube above the disc (side farther from balloon) to determine the stoma tract length. The measurement accuracy of the device is +/- 0.2cm.

CAUTION: An accurate measurement of the stoma length is critical for safety and comfort of the patient. An inappropriately sized device can cause necrosis, buried bumper syndrome, and/or hypergranulation tissue.

- To deflate the balloon, place fingers as shown below. The thumb should be placed directly below inflation port, while index finger is placed directly above inflation port. Gently squeeze to remove air from balloon. A noticeable sound may be heard when air escapes.

NOTE: Squeezing too hard will block the valve port, preventing balloon deflation. Only a partial squeeze - enough to open the valve - is all that is needed.

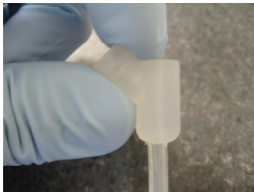
NOTE: Typical deflation using a slip tipped syringe can also be used to deflate the balloon.

- Gently remove the Measuring Device.

NOTE: The device can be disposed of by following local disposal guidelines, facility protocol or through conventional waste.

- Refer to the manufacturer's Directions for Use for the selection of the feeding tube and instructions for placement.

THANK YOU! Thank you for choosing AMT. For additional help and information regarding the use of our device, feel free to contact AMT with the contact information on the back of the instructions or use. We are happy to hear your thoughts and help with your concerns and questions.

R_x ONLY Prescription Only	STERILE EO Sterilized Using Ethylene Oxide
 Not made with natural rubber latex	 Not made with DEHP (Di(2-ethylhexyl) Phthalate)
 Do not resterilize	 Do not use if package is damaged and consult instructions for use
 Single Use Only	MD Medical Device

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© 2024 Applied Medical Technology, Inc.
Applied Medical Technology, Inc.