AMT MiniACE[®] Irrigation Set

CE
2862

ES	AMT MiniACE [®] Juego de Irrigación.
DE	AMT MiniACE [®] Irrigationsset.
IT	AMT MiniACE [®] Set d'irrigazione.
FR	AMT MiniACE [®] Kit d'irrigation.
NO	AMT MiniACE [®] irrigasjonssett.
SV	AMT MiniACE [®] Spolset.
PT-BR	AMT MiniACE [®] Conjunto de Irrigação.
NL	AMT MiniACE [®] Irrigatieset.
DA	AMT MiniACE [®] Udskylningssæt.
FI	AMT MiniACE [®] Kastelusarja.
AR	AMT MiniACE [®] مجموعة إرواء.
BG	AMT MiniACE [®] Иригационен комплект.
CS	AMT MiniACE [®] Vyplachovací sada.
ET	AMT MiniACE [®] irrigatsioonimoodul.
EL	AMT MiniACE [®] Σετ έκπλυσης.
HU	AMT MiniACE [®] Irrigáló szett.
JA	AMT MiniACE [®] 洗浄セット。
LV	AMT MiniACE [®] Irigācijas komplekts.
LT	„AMT MiniACE [®] “ Irigacijos rinkinys.
PT	AMT MiniACE [®] Conjunto de Rego.
PL	AMT MiniACE [®] Zestaw do irygacji.
RO	AMT MiniACE [®] Set de irigare.
SK	AMT MiniACE [®] Zavlažovacia/vyplachovacia súprava.
SL	AMT MiniACE [®] Komplet za izpiranje.
TR	AMT MiniACE [®] İrrigasyon Seti.

Rx Only

C4632-E 01/2024

Made in the U.S.A.

FIG A

FIG B — MiniACE® Button


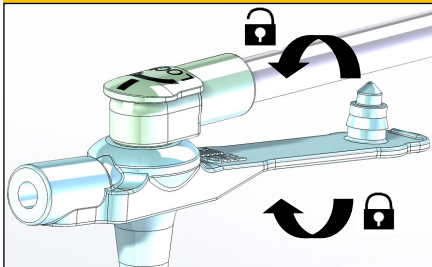
FIG C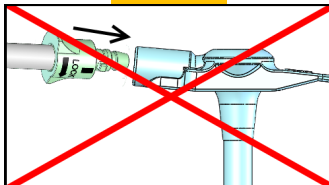

INDICATIONS, CONTRAINDICATIONS

INTENDED USE

An effective device used in channeling irrigation fluid from a fluid source to a bowel management device. For use in patients of all ages.

INDICATIONS FOR USE

The Bowel Management Irrigation Set is intended to allow connection between a compatible bowel management device and the delivery tubing of an irrigation/enema bag system.

CONTRAINDICATIONS

At this time there are no contraindications for use of the Irrigation Sets.

CLINICAL BENEFITS

Clinical benefits to be expected when using the Irrigation Set include but are not limited to:

- Provides connection between irrigation/enema bags and bowel management device.
- Allows for channeling of irrigation fluid to support daily continence.

PERFORMANCE CHARACTERISTICS

Performance characteristics of the Irrigation Set include but are not limited to:

- Provides secure attachment to bowel management device.
- Safety port cover and clamp provide additional leakage protection.

IRRIGATION SET INSTRUCTIONS

WARNING: THIS DEVICE HAS THE POTENTIAL TO MISCONNECT WITH SMALL BORE CONNECTORS OF OTHER HEALTHCARE APPLICATIONS. ONLY USE DEVICE TO CONNECT TO COMPATIBLE IRRIGATION DEVICES.

NOTE: This device is intended to be used by qualified professionals and lay users.

1. Inspect contents for damage. If damaged, do not use. Obtain another package.
2. The irrigation set may be used for gravity irrigation/enema.
3. Ensure the clamp is closed and attach the irrigation set (**Fig A**) to the MiniACE[®] button by lining up the dark line on the irrigation set connector with the dark line of the interlock on the MiniACE[®] button. Fully press the irrigation set connector into the MiniACE[®] button. Turn 3/4 to the right (clock-wise) to lock the irrigation set in place (**Fig B**).
4. Attach the opposite end of the irrigation set to the connector being used. If using a bolus or luer connector, firmly insert the connector while using a slight rotation into the irrigation set to lock in place. If using a rotating connector, firmly rotate the connector clockwise into the irrigation set while avoiding excessive force to lock in place. Once connected, open the clamp to allow flow.

WARNING: ONLY TIGHTEN BY HAND. NEVER USE EXCESSIVE FORCE OR A TOOL TO TIGHTEN A ROTATING CONNECTOR. IMPROPER USE CAN LEAD TO CRACKING, LEAKAGE, OR OTHER FAILURE.

WARNING: ENSURE DEVICE IS CONNECTED TO AN IRRIGATION PORT ONLY AND NOT TO AN IV SET.

WARNING: IF IRRIGATION SET IS NOT PROPERLY PLACED AND LOCKED, LEAKAGE MAY OCCUR. WHEN TWISTING THE CONNECTOR HOLD THE MINIACE® BUTTON IN PLACE TO AVOID ROTATION WHILE PLACING THE IRRIGATION SET. NEVER INSERT IRRIGATION SET CONNECTOR INTO A BALLOON INFLATION PORT (FIG C); THIS MAY CAUSE THE INTERNAL BALLOON TO DEFLATE OR LEAD TO BALLOON FAILURE IF IMPROPER CONTENTS ARE INSERTED INTO THE BALLOON. REFER TO BUTTON DIRECTIONS FOR USE.

5. When irrigation is complete, flush with 5-10ml water (if not using water for irrigation). Close the clamp to prevent leakage while removing the irrigation set. Remove the irrigation set by holding the MiniACE® button in place, grasp the irrigation connector, twist 3/4 turn to the left (counter clock-wise) to unlock, and gently remove the irrigation set from the MiniACE® button (Fig B). Snap the MiniACE® button plug in place to keep lumen clean.
6. The irrigation set should be cleaned with soap and water and thoroughly rinsed. Store in a clean, dry place. Never use a dishwasher for cleaning.

WARNING: IN CASE OF FEVER, ABDOMINAL DISTENTION, INFECTION, BLOCKAGE, OR TISSUE NECROSIS, PATIENTS SHOULD SEE THEIR PHYSICIAN IMMEDIATELY.

WARNING: DO NOT PLACE ANY FOREIGN OBJECTS INTO THE IRRIGATION PORT.

WARNING: IF USING A TRANSITION CONNECTOR, ENSURE THAT IT IS NOT CONNECTED TO AN IV SET PRIOR TO USE.

WARNING: WHEN USING A BOLUS STYLE IRRIGATION SET, THIS DEVICE HAS THE POTENTIAL TO MISCONNECT TO BREATHING SYSTEM, LIMB CUFF, AND NEURAXIAL CONNECTORS.

NOTE: The ENFit® connection is intended to reduce misconnections with other non-enteral healthcare applications.

DEVICE LONGEVITY

Irrigation sets are meant to be periodically replaced for optimal performance and cleanliness. AMT recommends that the irrigation set be changed at least every 2 weeks or as often as indicated by your healthcare professional. Device performance and functionality can degrade over time depending on usage and environmental conditions. Some factors that can lead to reduced longevity include: intestinal pH, irrigation solution used, frequency of use, trauma to the device, and overall irrigation set care.

Device should be replaced if leakage, cracks, tears, excessive residue build-up, mold, fungus, or other signs of failure are noted. The device can be disposed of by following local disposal guidelines or through conventional waste. Some usage types can wear down device components quicker than others. If tubing is becoming hardened, replacement is recommended in near future to avoid failure.

NOTE: Please contact AMT, our Authorized Representative (EC Rep), and/or the competent authority of the member state in which you are established if a serious incident has occurred in relation to the device.

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© 2024 Applied Medical Technology, Inc.
ENFit[®] is a registered trademark of Global Enteral Device Supplier Association, Inc.
Applied Medical Technology, Inc.